# Mini Blade<sup>XT</sup>

## Instructions for Use

MBXTAB1SM-MBXTAP3SM

EN	Instructions for Use	2
FR	Instructions d'utilisation	12
DE	Gebrauchsanweisung	22
IT	Istruzioni per l'uso	32
ES	Instrucciones de uso	42
NL	Gebruiksaanwijzing	52
PL	Instrukcje użytkowania	62
PT	Instruções de utilização	72
CS	Návod k použití	82

# Contents

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	4
3 Construction .....	5
4 Function.....	5
5 Maintenance.....	6
6 Limitations on Use .....	6
7 Alignment .....	7
7.1 Static Alignment.....	7
7.2 Dynamic Alignment .....	7
8 Fitting Advice.....	8
9 Heel Spring Removal & Replacement .....	8
10 Technical Data .....	9
11 Ordering Information .....	10

# 1 Description and Intended Purpose

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to Mini BladeXT.

Ensure that the user understands all instructions for use, drawing particular attention to the maintenance and safety information.

## Application

This device is to be used exclusively as part of a lower limb prosthesis.

Intended for a single user.

This device is suitable for walking and is designed specifically for general sport and running.

## Activity Level

This device is recommended for users that have the potential to achieve Activity Level 3 or 4. Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

### Activity Level 3

Has the ability or potential for ambulation with variable cadence.

Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

### Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

## Contraindications

This device may not be suitable for Activity Level 1 and 2 individuals.

## Clinical Benefits

- Allows variable running speeds
- Sole plate improves perceived stability, compared to other Running-specific prostheses
- Users demonstrate confidence in prosthetic loading during high activity

## Spring Set Selection

Activity	User Weight kg (lbs)		
	4	20–35 (44–77)	36–50 (78–110)
Toe and Heel Spring Set			
1		2	3
Soft		Medium	Firm

## 2 Safety Information

**This warning symbol highlights important safety information which must be followed carefully.**

**Any changes in the performance or function of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.**

**Always use a hand rail when descending stairs and at any other time if available.**

**The user should be advised that because of the deep 'C' shape side profile (A-P profile) of this device, care should be taken when descending stairs and steps due to the potential hazard of 'over-stepping'.**

**Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.**

**The user should contact their clinician if their condition changes.**

**Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.**

**To minimize the risk of slipping and tripping, the sole cover should not be excessively worn, torn or loose.**


**Be aware of finger trap hazard at all times.**

# 3 Construction

## Principal Parts

- Heel & Toe Springs
  - Heel attachment Screws & Washers
  - Sole Cover & Wedge
  - Mini Pyramid
  - Pyramid Washer and Nut
- E-carbon  
Stainless steel, silver  
PU (polyurethane)  
Titanium  
Stainless steel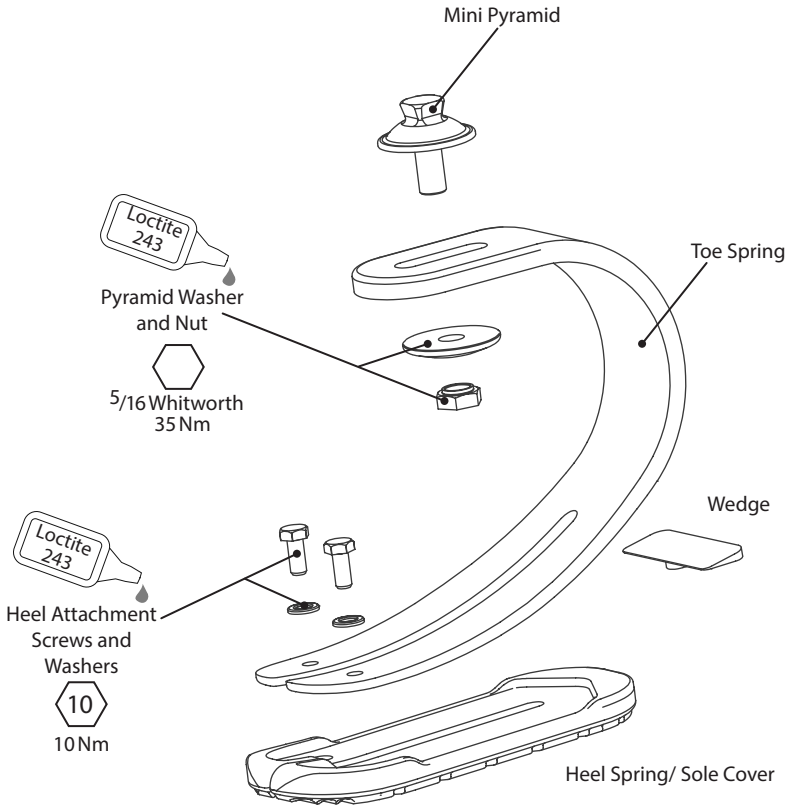

# 4 Function

This device comprises an e-carbon toe spring with an independent heel spring which is attached via two stainless steel screws and washers.

The heel spring has an over molded PU sole cover. No foot shell or shoe is required for this device.

---

## 5 Maintenance

Maintenance must be carried out by competent personnel.

We recommend the following annual maintenance:

- Check all screws for tightness, clean and reassemble as necessary.
- Check heel and toe springs for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user should be advised:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.
- Excessive wear of the sole cover could lead to loss of grip. The limb should not be used if the sole cover is excessively worn, torn or loose.
- Excessive wear or damage to the spring elements could lead to structural failure.

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

The user should be advised that a regular visual check of the foot is recommended, signs of wear that may affect function should be reported to their service provider.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

---

## 6 Limitations on Use

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).

Suitable for submersion

## 7 Alignment

### 7.1 Static Alignment

#### Trans-femoral Alignment


Align trans-femoral components according to fitting instructions supplied with the knee, keeping the build line relative to the device, as shown.

#### Set-up Length

This device should be set up with a 5 mm heel wedge so that the top surface is horizontal. The length of this device should be 5–10 mm longer than the user's everyday prosthesis with running trainer footwear to allow for spring compression.

#### Build Line

The build line should fall in the middle of the slot on top of this device, this is defined by a vertical alignment indicator line on each side.

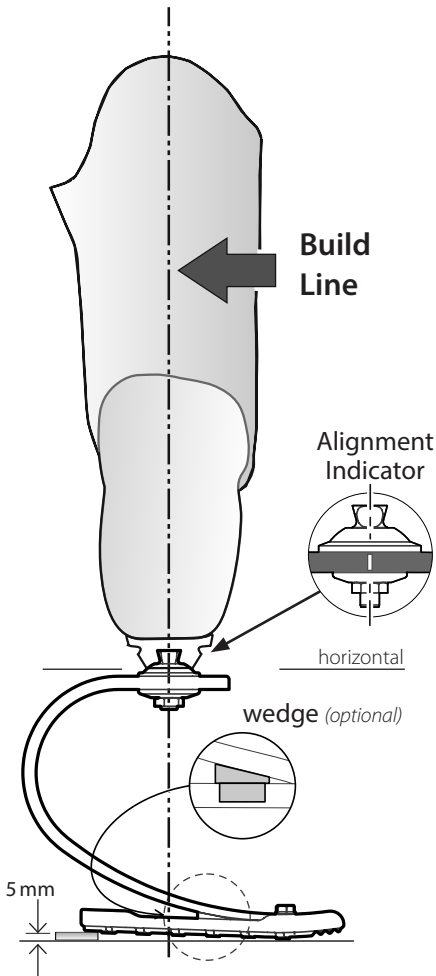Ensure the top surface of the device is horizontal to allow for compression in the main toe spring when loaded with the user's body weight.

### 7.2 Dynamic Alignment

#### Coronal Plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

#### Sagittal Plane

On standing the toe should deflect so that the heel is in contact with the ground. On walking, check for a smooth transition from heel strike to toe off.

1. Optimize plantar flexion and dorsiflexion  
On running, a heel strike should always be present but this can be very small and subtle. To optimize running gait, plantarflex the foot incrementally in small amounts at a time ensuring a heel strike is maintained. If the foot is too plantar flexed the sole will 'slap' against the ground. To prevent 'slap' dorsiflex the foot.
2. Stiffness and A-P shift  
When an optimal plantar flexion/dorsiflexion angle has been achieved, if when running the toe feels too soft, then shift the weight line posteriorly on the foot. If the toe feels too stiff, then shift the weight line anteriorly on the foot.
3. If the user is sinking at heel strike the heel is too soft. Fitting the wedge supplied will have the effect of stiffening the heel spring, this can be taped in place for trial (see diagram). For permanent fitting the wedge should be glued in place by application of Loctite 424 (926104) on the upper surface of the wedge.

## 8 Fitting Advice

Springs are supplied as matched sets i.e. the shin/toe and heel springs are designed to work together to give smooth progression for most users.

Condition	Symptoms	Remedy
Heel too soft	Sinking at heel strike	Add Heel Wedge
Heel too hard	Rapid transition from heel strike through stance phase Difficulty in controlling heel action, foot jars into mid-stance	1. Remove Heel Wedge (if fitted) 2. Order a softer Heel Spring category
Toe too soft	Rapid progression through mid-stance 'Drop-off' during running	1. Move socket posteriorly in relation to foot 2. Plantar flex foot slightly - note: heel strike must be maintained
Toe too hard	Feels like 'climbing' over toe	1. Move socket anteriorly in relation to foot 2. Dorsiflex foot slightly

Please contact your supplier if it is not possible to achieve a smooth gait after following the recommendations above.

## 9 Heel Spring Removal & Replacement

**Use appropriate health and safety equipment at all times including extraction facilities.**

**Be aware of finger trap hazard at all times.**


---

# 10 Technical Data

Material:	E-carbon, PU , titanium, St. Stl.
Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight:	450 g (1 lb)
Activity Level:	4
Maximum User Weight:	60 kg (132 lb)
Proximal Alignment Attachment:	Male Mini Pyramid (Blatchford)
Range of Adjustment:	360° rotation ±5° angular 30 mm A-P shift
Build Height:	168 mm
Proximal pyramid to ground level (See diagram below)	

---

## Fitting Length

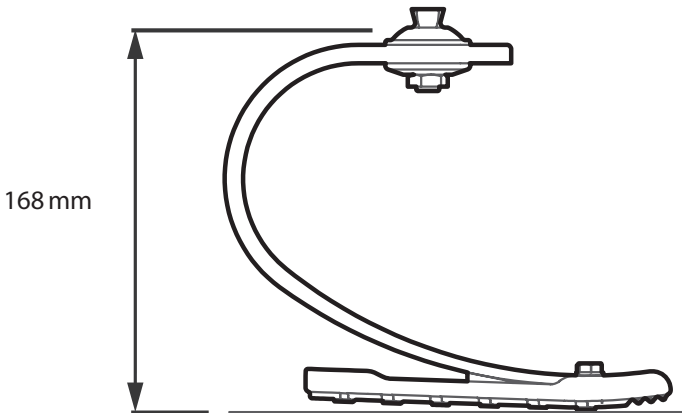---

# 11 Ordering Information

## Order Example

<b>MBXTA</b>	<b>B</b>	<b>3</b>	<b>SM</b>
--------------	----------	----------	-----------

B - Blue/ Spring Set  
P - Pink 1,2 or 3

e.g. MBXTAB3SM

## MBXT Heel Spring Kit

Includes the overmoulded cover, fasteners and heel wedge.

Spring category	Heel type	Part no.
1	Soft	539084SS
2	Medium	539084MS
3	Firm	539084FS

## Proximal Adapter

Item	Part No.
Male Mini Pyramid Kit	189627

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Blatchford mini pyramid components must only be used in conjunction with other Blatchford mini pyramid components.

## Warranty

This device is warranted for 12 months (excludes paintwork and sole cover). The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions. See Blatchford website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

See Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

## Retaining the Packaging Label

You are advised to keep the packaging label as a record of the device supplied.

## Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

## Trademark Acknowledgements

BladeXT and Blatchford are registered trademarks of Blatchford Products Limited.

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**Blatchford Europe GmbH**
**Endolite India Ltd.**