ESK+ 4-Bolt

Instructions for Use

ESK4BP30
ESK4BPPYR

SMARTIP4B30
SMARTIP4BPYR

ESK4B30
ESK4BPYR

| | | |
|----|-----------------------|-----|
| EN | Instructions for Use | 2 |
| DA | Brugsanvisning | 19 |
| NO | Bruksanvisning | 36 |
| FI | Käyttöohjeet | 53 |
| SV | Bruksanvisning | 70 |
| RU | Инструкция протезиста | 87 |
| ZH | 使用说明 | 104 |
| AR | إرشادات الاستخدام | 121 |

| | |
|--|----|
| Contents | 2 |
| 1 Description and Intended Purpose | 3 |
| 2 Safety Information | 5 |
| 3 Construction | 6 |
| 4 Function..... | 7 |
| 5 Maintenance | 7 |
| 6 Limitations on Use | 8 |
| 7 Static Alignment | 9 |
| 8 Alignment Advice and Stability Adjustment..... | 10 |
| 9 Front Stop Adjustment - Adjustable Trunnion..... | 11 |
| 9.1 Further Adjustments..... | 12 |
| 10 Assembly Instructions | 13 |
| 10.1 Changing Front Stop Buffer..... | 13 |
| 10.2 Changing Shin Stops..... | 13 |
| 10.3 Removing ESK+ Mechanism | 14 |
| 10.4 Replacement of ESK+ Mechanism | 15 |
| 11 Technical Data | 16 |
| 12 Ordering Information | 17 |

1 Description and Intended Purpose

These Instructions for Use are intended for use by the practitioner and user unless otherwise stated.

The term device is used throughout these instructions for use to refer to 4-Bolt ESK+ products.

Please read and ensure you understand all instructions for use, in particular all safety information and maintenance instructions.

Application

This device is to be used exclusively as part of a lower limb prosthesis.

This device is for a single user.

This device is a knee unit that provides a uniaxial frame for a weight activated stabilized knee which provides support for the prosthesis at various cadences on a variety of terrains and is intended to improve safety and reduce compensation.

The shin can be fitted with a PSPC (Pneumatic Swing Phase Control), a SmartIP (Smart Intelligent Prosthesis) or supplied without a cylinder.

The device has a choice of 4-bolt or T-slot connections proximally and a choice of male pyramid or 30 mm tube clamp connections distally.

Features

| | Manually adjusted PSPC | Microprocessor controlled SmartIP |
|--|---------------------------|--------------------------------------|
| Stance flexion during weight bearing | ✓ | ✓ |
| Tough carbon fibre frame | ✓ | ✓ |
| 140 degree knee flexion | ✓ | ✓ |
| Adjustable sensitivity of knee stability | ✓ | ✓ |
| Independent manually adjustable swing controls for flexion and extension | ✓ | |
| Microprocessor controlled swing control in flexion with manually adjustable terminal impact damping in extension | | ✓ |

Activity Level

This device is for Activity Level 2, 3 and 4 users (weight limits apply, see *Technical Data*).

This device might not be suitable for Activity Level 1 users or for use in competitive sports events. These types of users might be better served by a specially designed prosthesis that is optimized for their needs.

Activity Level 1

Has the ability or potential to use a prosthesis for transfers or ambulation on level surfaces at fixed cadence. Typical of the limited and unlimited household ambulator.

Activity Level 2

Has the ability or potential for ambulation with the ability to traverse low-level environmental barriers such as curbs, stairs, or uneven surfaces. Typical of the limited community ambulator.

Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

Clinical Benefits

- Weight-activated stance control and adjustable swing control provide smooth, safe gait with a free knee.
- Microprocessor swing control allows optimized gait characteristics across a range of walking speeds. (SmartIP only.)
- The stabilizing mechanism has a resilient action which provides limited stanceflexion under load to absorb shock loads at heel strike.

Contraindications

This device is not suitable for:

- Users with extreme muscular weakness, contractures that cannot be correctly accommodated, or proprioceptive dysfunction including poor balance.
- Users with contralateral joint instabilities or pathology.
- Users with complicated conditions involving multiple disabilities.
- Users, including bilateral transfemoral amputees, who require a yielding function from the knee to descend stairs or to sit. This can result in damage of the brake mechanism that can adversely affect stability and durability.

Users must be given gait training before using this device.

Make sure that the user has understood all instructions for use, drawing particular attention to all maintenance and safety information sections.

2 Safety Information

This warning symbol highlights important safety information.

Beware of finger trap hazard all times.

Any changes in the performance or function of the limb e.g. instability, double-action, restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

Any excessive changes in the heel height after finalization of alignment may adversely affect limb function.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

Care should be taken when carrying heavy loads as this may adversely affect the stability of the device.

The user should be advised to contact their practitioner if their condition changes.

Avoid exposure to extreme heat and/or cold.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.


The device is not intended for use when immersed in water or as a shower prosthesis. If the limb comes into contact with water wipe dry immediately.

3 Construction

Principal Parts

- Carrier Assembly Carbon fibre composite & stainless steel
- Chassis Assembly Aluminium alloy, stainless steel, engineering thermoplastic, fibre reinforced (asbestos free) friction material.
- Distal Dome Aluminium alloy
- Distal Pyramid Titanium with stainless steel bolt
- Distal Tube Clamp Aluminium alloy
- Patella Engineering thermoplastic
- PSPC Aluminium alloy, stainless steel, titanium, engineering thermoplastic
- SmartIP Control Cylinder Aluminium alloy, stainless steel, Titanium, engineering thermoplastic
Electronic components, batteries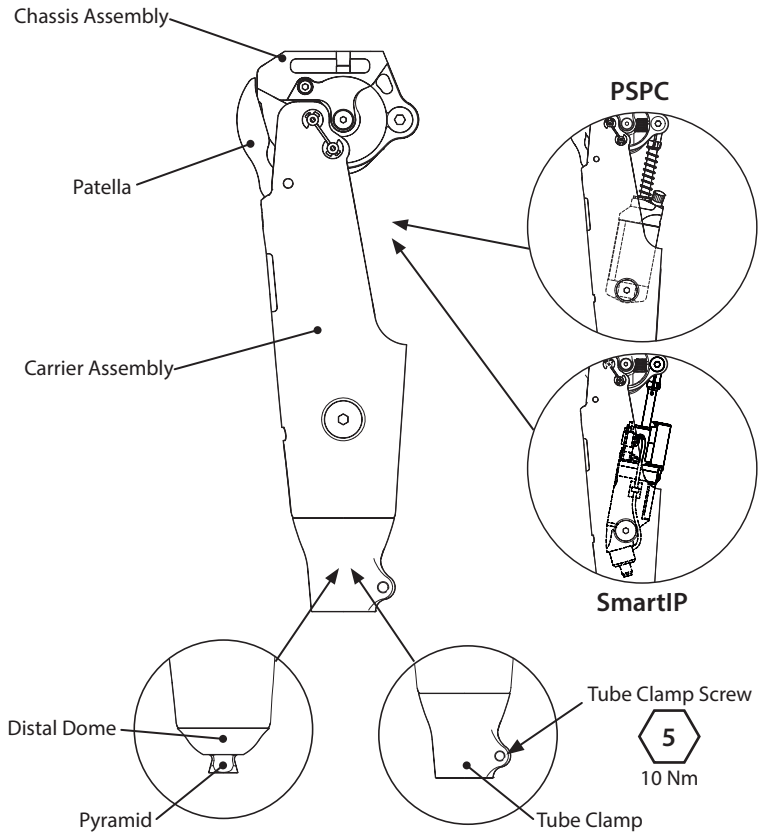

4 Function

The knee is a uniaxial design comprising a knee pivot connecting a chassis to a carrier. A weight activated stabilising knee mechanism is housed within the chassis. The stabilising mechanism has a resilient action which provides limited stance flexion under load to absorb shock loads at heel strike.

A knee control cylinder sits within this frame assembly, fitted between the chassis posterior pivot and the carrier distal pivot.

The PSCP cylinder has manually adjustable valves which give independent control of flexion and extension resistance during swing phase.

The SmartIP control cylinder uses sensors inside the cylinder to monitor walking speed and automatically adjusts flexion resistance accordingly. Extension control is via fixed adjustment of a terminal impact damper.

5 Maintenance

Visually check the device regularly.

Report any changes in performance of this device to the practitioner/service provider e.g. unusual noises, increased stiffness or restricted/excessive rotation, significant wear or excessive discoloration from long term exposure to UV.

Inform the practitioner/service provider of any changes in body weight and/or activity level.

Cleaning

Use a damp cloth and mild soap to clean the outside surfaces. DO NOT use aggressive cleansers.

The remaining instructions in this section are for practitioner use only.

This maintenance must be carried out only by competent personnel (practitioner or suitable trained technician).

To maintain optimum performance, a periodic inspection for unacceptable noise, play and stability is recommended. The period between inspections is determined by factors such as the activity of the user, frequency of usage and the stability setting of the knee.

As a guide, if the setting is 6 clicks or above and the user averages 600,000 steps per annum, an annual inspection is recommended.

If the stability setting is less than 6 clicks and/or the user averages fewer than the average number of steps, the period in between inspections may be longer.

The use of a Limb Activity Monitor is recommended to determine step count.

We recommend adjusting the knee to the individuals' stability requirements to ensure optimal stability and durability. See adjustment guide/flow diagram.

At these inspections ;

- Check for visual defects that may affect proper function.
- Check front stop and replace if necessary.
- Check shin stops and replace if necessary.
- Dis-assemble, clean and reassemble.

Advise the user that a regular visual check of the device is recommended and any changes in performance of this device must be reported to the practitioner. Changes in performance may include: increase or decrease in knee stiffness, instability, reduced knee support (free movement), any unusual noise.

Storage and Handling

When storing for prolonged periods, the device should be kept dry and within a temperature range of -15 °C to 50 °C. (5 °F to 122 °F)

Use product packaging supplied.

6 Limitations on Use

Intended Life:

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

Avoid exposing the device to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).


Suitable for outdoor use

7 Static Alignment

The instructions in this section are for practitioner use only.

Walking Trials

Spring Stack Adjuster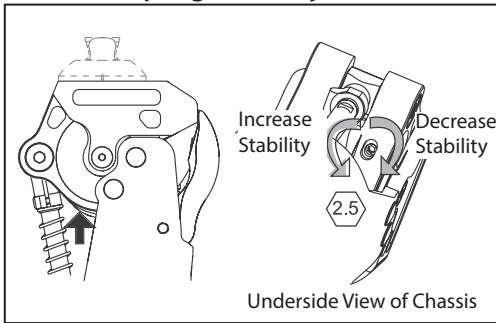

Fully extend limb to access the adjuster.


Rotate the **SPRING STACK ADJUSTER** to achieve the required stability for the individual amputee.

Note... Audible clicks should be heard upon adjustment, and the adjuster is self locking.

Tighten the **SPRING STACK** to reduce stability.


Release the **SPRING STACK** to increase stability
- See above.

If required, the adjuster may be reset to the factory setting. This can be achieved by reducing stability by six audible clicks from the maximum stability setting.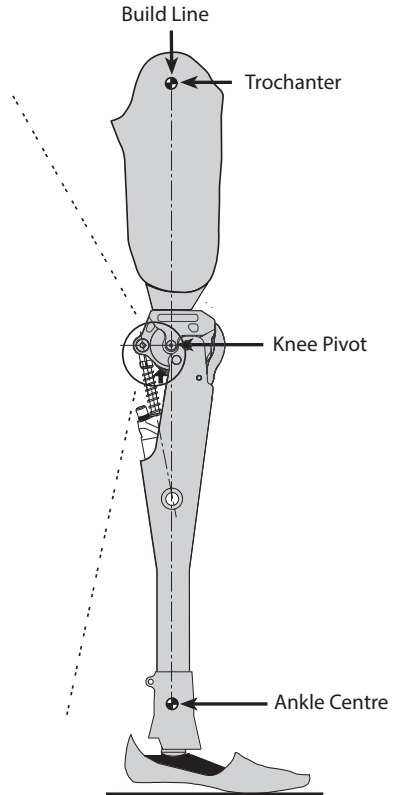

8 Alignment Advice and Stability Adjustment





The instructions in this section are for practitioner use only.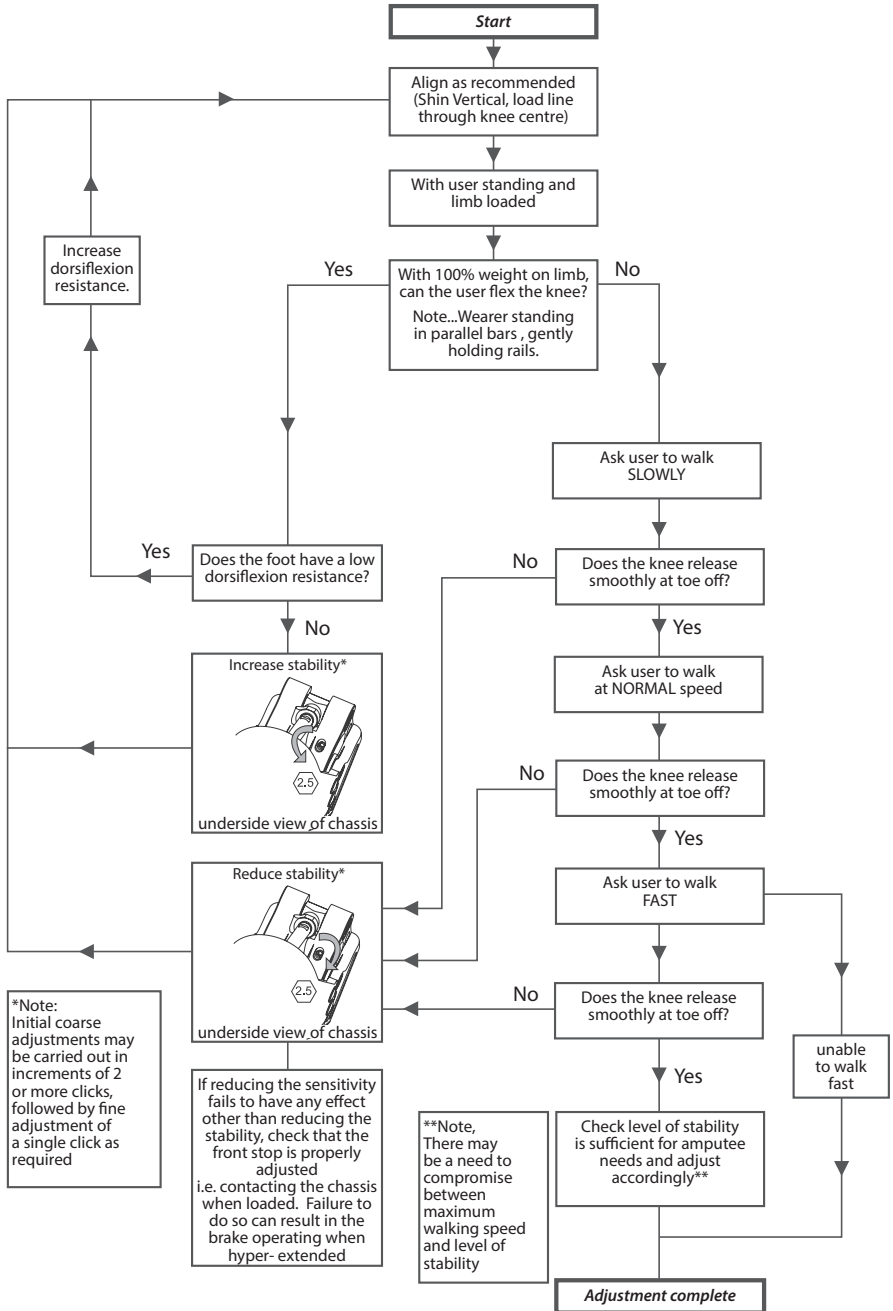

9 Front Stop Adjustment - Adjustable Trunnion

The instructions in this section are for practitioner use only.

Fitted to the PSPC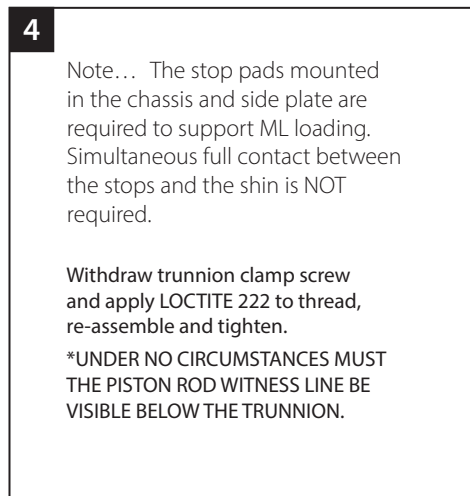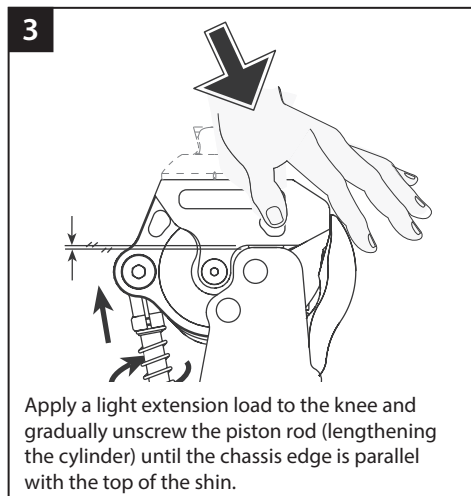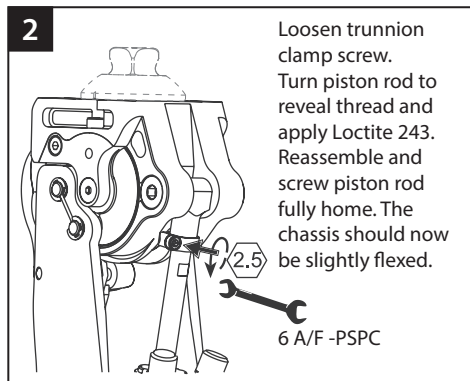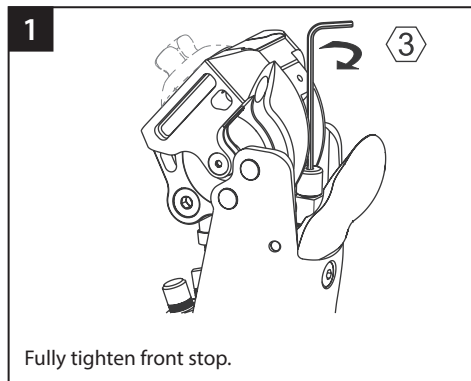

Front Stop Adjustment - Non-Adjustable Intelligent Prosthesis

(As fitted to older SmartIP models.)

No adjustment is necessary when using these cylinders. They require the front stop to be fully tightened as shown in Stage 1.

For adjustment of swing control devices see appropriate IFU.

9.1 Further Adjustments

The instructions in this section are for practitioner use only.

| Symptom | Solution |
|---|--|
| Knee does not stabilise | Increase sensitivity |
| | Shift socket Anteriorly |
| | Replace stabilizing mechanism if trailing edge of brake is damaged |
| Knee does not release | Decrease sensitivity |
| | Check correct adjustment of piston rod length |
| | Shift socket posteriorly |
| Heel rise too high during swing | See control cylinder IFU |
| Heel rise too low during swing | See control cylinder IFU |
| User experiences terminal impact | See control cylinder IFU |
| Device does not fully extend during swing | See control cylinder IFU |



If after following these instructions you still have problems with the function please contact the sales team in your area for advice.

10 Assembly Instructions



The instructions in this section are for practitioner use only.

10.1 Changing Front Stop Buffer

When fitting a new front stop, always re-adjust, and if applicable check length of cylinder if fitted with an adjustable trunnion. Adjust if necessary. (See previous page.)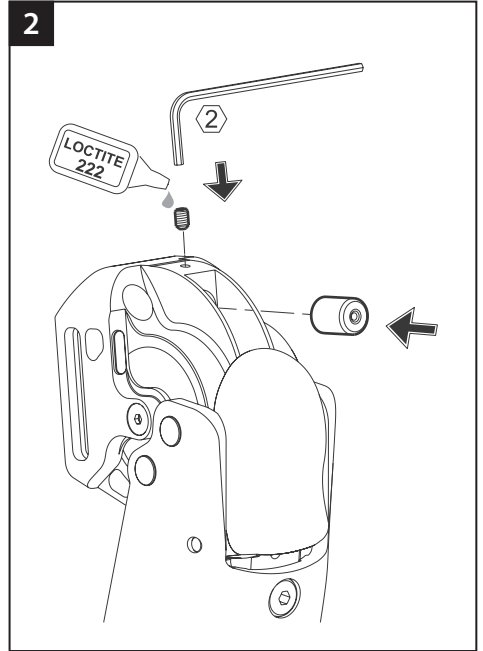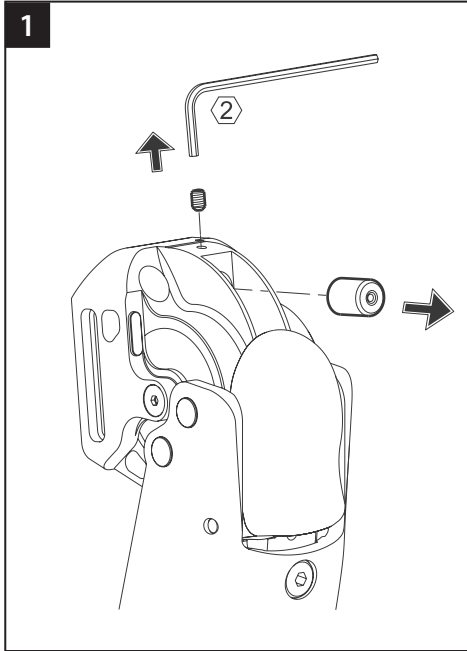


10.2 Changing Shin Stops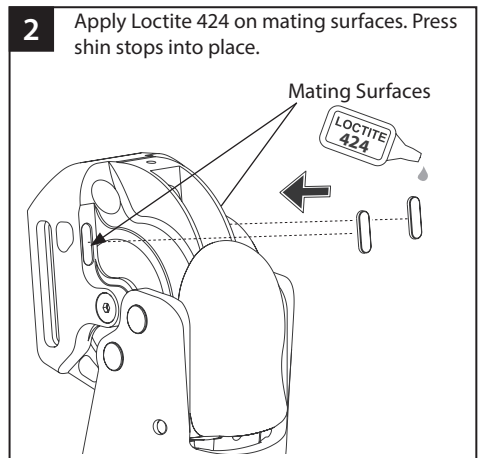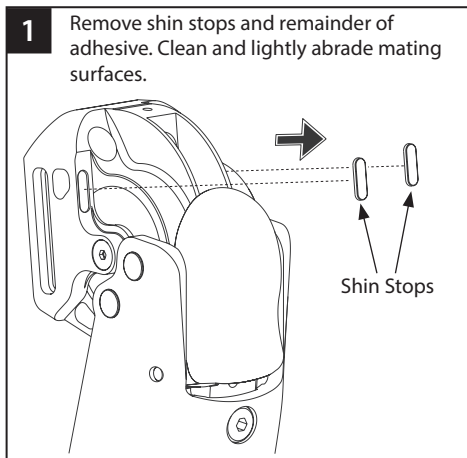

10.3 Removing ESK+ Mechanism


The instructions in this section are for practitioner use only.

1 Remove all preload and record number of clicks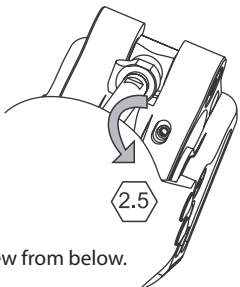


View from below.


2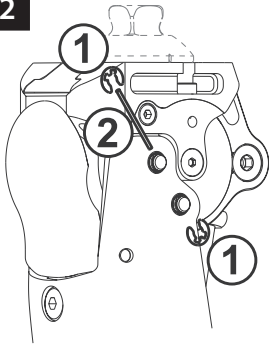


3

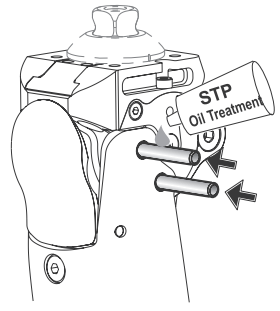
4

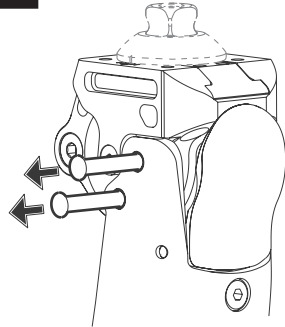
5

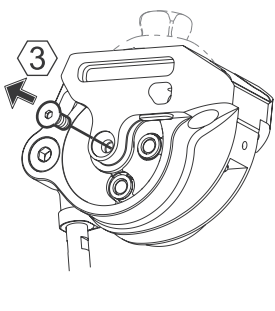6

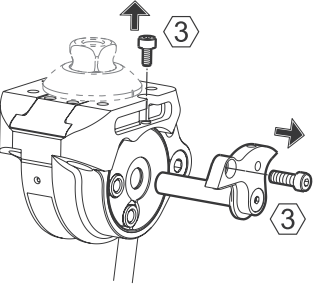
7

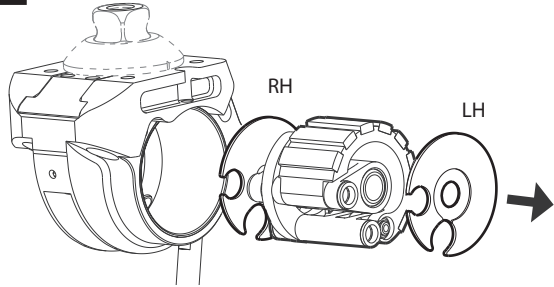Brake band material may be wiped clean with paper, do not use solvents.








8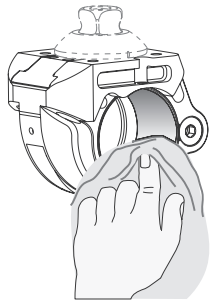

Clean bore with suitable solvent

10.4 Replacement of ESK+ Mechanism


The instructions in this section are for practitioner use only.
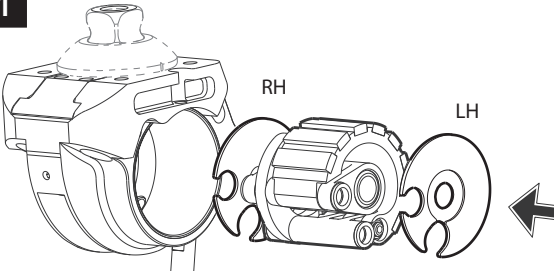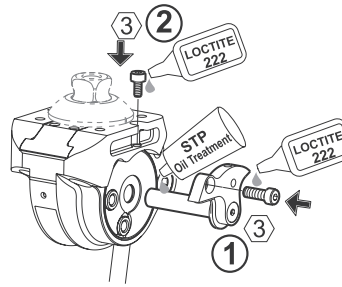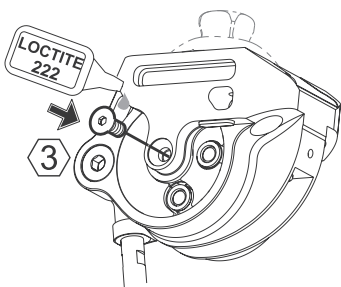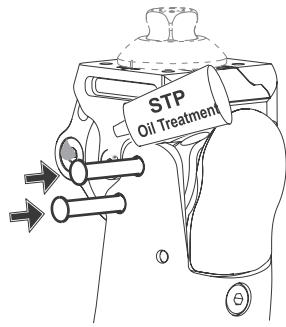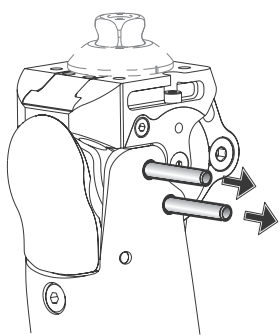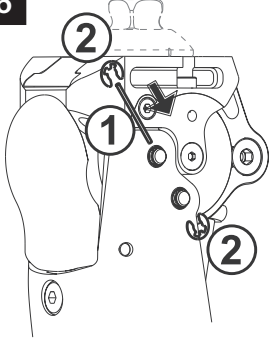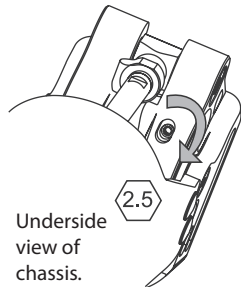
| | | |
|---|--|---|
| <p>1</p>  <p>ORIENTATION IS IMPORTANT! Assemble flat sides and lettering (RH, LH) facing inwards.</p> | <p>2</p>  | |
| <p>3</p>  | <p>4</p>  | |
| <p>5</p>  | <p>6</p>  | <p>7 Re-apply preload with recorded number of clicks.</p>  <p>Underside view of chassis.</p> |


11 Technical Data

| | |
|--|--|
| Operating and Storage Temperature Range: | -15 °C to 50 °C (5 °F to 122 °F) |
| Component Weight: | |
| SmartIP | 1.27 kg (2 lb 9 ozs) |
| PSPC | 1.13 kg (2 lb 8 ozs) |
| Activity Level: | 2-3 (4) |
| Maximum User Weight: | |
| Activity Levels 2-3 | 125 kg (275 lb) |
| Activity Level 4 | 100 kg (220 lb 7.4 ozs) |
| Maximum Flexion Angle: | 140° |
| Proximal Alignment Attachment: | 4-bolt or T slot |
| Distal Alignment Attachment: | 30 mm Tube Clamp or Male Pyramid (Blatchford) |
| Build Height: | See Diagrams Below. |

Fitting Length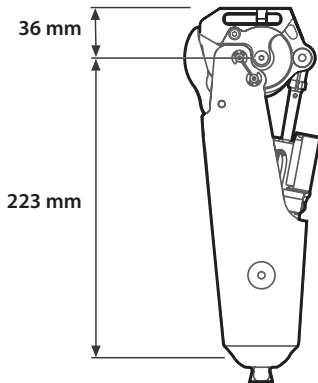

**ESK+ 4-Bolt
with Distal Pyramid**

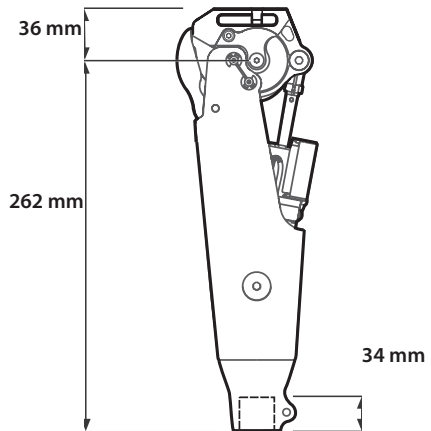**ESK+ 4-Bolt
with 30 mm Distal Tube Clamp**

12 Ordering Information

Order Example

| | ESK+ with 4-Bolt Adapter | | | | | | | |
|----------------------------|--------------------------|----------------|-------------------------|-------------------------|----------------|----------------|----------|----------------|
| | 30 mm Distal Tube Clamp | | | | Distal Pyramid | | | |
| Pneumatic Cylinder | ESK | 4B | P | 30 | ESK | 4B | P | PYR |
| | ESK+ | 4-Bolt Adapter | PSPC | Distal Tube Clamp 30 mm | ESK+ | 4-Bolt Adapter | PSPC | Distal Pyramid |
| SmartIP Knee | SMARTIP | 4B | 30 | SMARTIP | 4B | PYR | | |
| | SmartIP Knee | 4-Bolt Adapter | Distal Tube Clamp 30 mm | SmartIP Knee | 4-Bolt Adapter | Distal Pyramid | | |
| No Control Cylinder Fitted | ESK | 4B | 30 | ESK | 4B | PYR | | |
| | ESK+ | 4-Bolt Adapter | Distal Tube Clamp 30 mm | ESK+ | 4-Bolt Adapter | Distal Pyramid | | |

Replacement Parts

| Item | Part. No. |
|--------------------------------|-----------|
| 4-Bolt ESK+ Knee Assembly | 239150 |
| ESK Mechanism Alignment Pin | 234914 |
| Front Stop Pad | 234903 |
| ESK+ Stabilizing Mechanism Kit | 239949 |

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical Device

Single Patient – multiple use

Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

Warranty

This device is warranted for 24 months, the brake band for 12 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This product contains hydraulic oil, mixed metals and plastics. Where possible the components should be recycled in accordance with local waste handling regulations.

Retaining the Packaging Label

The practitioner is advised to keep the packaging label as a record of the device supplied.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

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Blatchford Europe GmbH
Endolite India Ltd.