Sliding/Rotating Male Pyramid 4-Bolt Adapter Kit

Instructions for Use

169407

| | | |
|----|-----------------------|----|
| EN | Instructions for Use | 2 |
| DA | Bruksanvisning | 11 |
| NO | Bruksanvisning | 20 |
| FI | Käyttöohjeet | 29 |
| SV | Bruksanvisning | 38 |
| RU | Инструкция протезиста | 47 |
| ZH | 使用说明 | 56 |
| AR | إرشادات الاستخدام | 65 |

Contents

| | |
|--|---|
| Contents | 2 |
| 1 Description and Intended Purpose | 3 |
| 2 Safety Information | 4 |
| 3 Construction | 5 |
| 4 Function..... | 6 |
| 5 Maintenance | 6 |
| 6 Limitations on Use | 7 |
| 7 Fitting Advice..... | 7 |
| 8 Technical Data | 8 |
| 9 Ordering Information | 9 |

1 Description and Intended Purpose

These Instructions for Use are intended for use by the practitioner and user unless otherwise stated.

The term *device* is used throughout these instructions for use to refer to Sliding/Rotating Male Pyramid 4-Bolt Adapter Kit.

Please read and ensure you understand all instructions for use, in particular all safety information and maintenance instructions.

Application

This device is an adapter to allow secure connection between a Blatchford female pyramid connector on a prosthetic limb and a prosthetic socket with a four-bolt connection. The device provides horizontal slide adjustment in the anterior-posterior plane, rotational and tilt adjustment in the vertical plane.

Intended for a single user.

Feature

- Lightweight, strong, aluminum and titanium construction

Activity Level

This device is recommended for users that have the potential to achieve Activity Levels 1-4 (weight limits apply, see *Technical Data* section). Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

Activity Level 1

Has the ability or potential to use a prosthesis for transfers or ambulation on level surfaces at fixed cadence. Typical of the limited and unlimited ambulator.

Activity Level 2

Has the ability or potential for ambulation with the ability to traverse low level environmental barriers such as curbs, stairs, or uneven surfaces. Typical of the limited community ambulator.

Activity Level 3

Has the ability or potential for ambulation with variable cadence.

Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

Contraindications

There are no known contraindications if this device is used in accordance with these instructions.

Clinical Benefits

- Allows connection of a Blatchford Pyramid System and a Blatchford 4-Bolt interface.
- Allows horizontal slide and rotation to optimise alignment.

2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Any changes in the performance should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

To reduce the risk of injury due to failure or loosening of the screw connections ensure the threads are cleaned thoroughly before each installation.

Always apply the specified torque values to screws. Always use the screw type specified.

Be aware of finger trap hazard when adjusting or maintaining the adapter kit.

Do not use the device if you notice any unusual noises or excessive play in the product.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified practitioner that has attended an approved training course.


Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

3 Construction

Principal Parts

- Alignment Kit Dome Aluminum alloy
- Male Pyramid Titanium alloy
- 4 X M6 Screws Stainless steel
- Adapter Plate Aluminum alloy

Component Identification

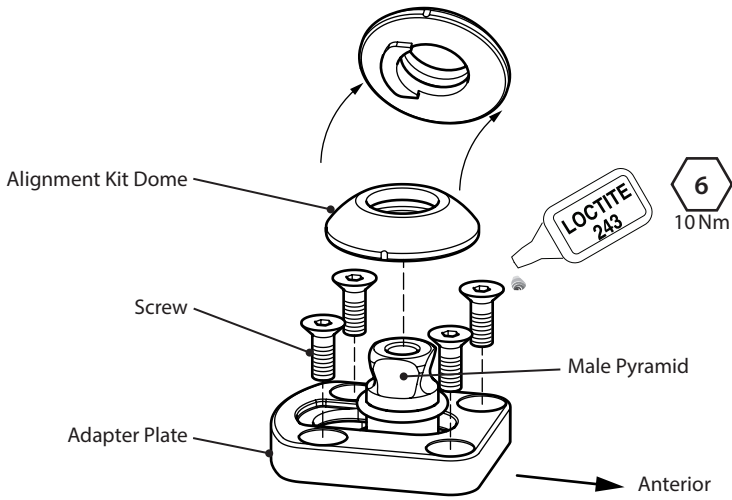4 Function

This device is an adapter to allow secure connection between a Blatchford female pyramid connector on a prosthetic limb and a prosthetic socket with a 4-bolt connection. The device provides horizontal slide adjustment in the anterior-posterior plane, rotational and tilt adjustment in the vertical plane.

When the adapter's 4-bolt connection screws are securely fastened to the socket, and the grub screws on the attaching limb's female pyramid are securely fastened to the adapter's male pyramid, then the device provides a secure connection between limb and socket.

For correct screw torque settings and Loctite application for secure connection, refer to the *Construction* section for the 4-bolt connection and refer to the Instructions for Use of the attaching Blatchford device for the pyramid connection.

5 Maintenance

Visually check the device regularly.

Report any changes in performance of this device to the practitioner/service provider e.g. unusual noises, loss of alignment or looseness.

Inform the practitioner/service provider of any changes in body weight and/or activity level.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers. Thoroughly dry before use.

The remaining instructions in this section are for practitioner use only.

This maintenance must be carried out only by competent personnel (practitioner or suitable trained technician).

The following routine maintenance is to be carried out at least annually:

- Check the alignment parts for tightness (see *Construction* section). If loose, remove and clean, then apply Loctite and tighten to the correct torque setting.
- Check for defects that could affect proper function.

Ensure the user has read and understood all safety and user-level maintenance information.

Advise the user that a regular visual check of the foot is recommended and signs of wear that may affect function should be reported to their service provider.

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

6 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof to a maximum depth of 1 meter. Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts. Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15 °C and 50 °C
(14 °F to 122 °F).

Suitable for submersion

7 Fitting Advice


The instructions in this section are for practitioner use only.

| Symptom | Solution |
|---|---|
| A recurring noise occurs between the socket and device. | Confirm all bolts are securely tightened to the torque ratings specified in this document. Apply Loctite. |
| The device moves out of position. | User must not use the device until adjusted, repaired or replaced. |

8 Technical Data

| | |
|--|--|
| Operating and Storage Temperature Range: | -15°C to 50°C (5°F to 122°F) |
| Component Weight: | 125 g (4.4 oz) |
| Activity Level: | 1–4 |
| Maximum User Weight: | 145 kg (4) 150 kg (1-3) (319 lb 11 oz) (330 lb 11 oz) |
| Range of Adjustment: | 360° rotation, vertical axis ±7° tilt, vertical axis ±10 mm horizontal shift, anterior-posterior axis |
| Attachment Types: | 4 x M6 x 16 mm screws (Blatchford) Male Blatchford Pyramid |
| Build Height: | 15 mm |

Fitting Length

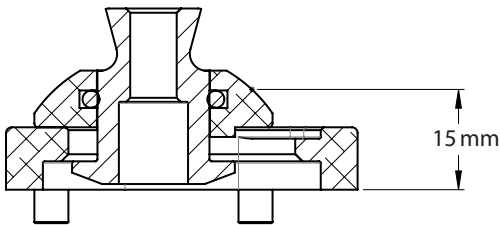9 Ordering Information

Order Number 169407

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical Device

Single Patient – multiple use

Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a practitioner.

Warranty

This device is warranted for 24 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See the Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This product is constructed from recyclable materials. Where possible the components should be recycled in accordance with local waste handling regulations.

Retaining the Packaging Label

The practitioner is advised to keep the packaging label as a record of the device supplied.

Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

Manufacturer's Registered Address

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Blatchford Europe GmbH
Endolite India Ltd.