# Mini Blade<sup>XT</sup>

## User Guide

MBXTAB1SM-MBXTAP3SM

EN	User Guide	2
FR	Guide de l'utilisateur	8
DE	Bedienungsanleitung	14
IT	Guida per il paziente	20
ES	Guía del usuario	26
NL	Gebruikershandleiding	32
PL	Podręcznik użytkownika	38
PT	Guia do utilizador	44
CS	Návod k obsluze	50

# Contents

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	3
3 Construction .....	4
4 Maintenance .....	5
5 Limitations on Use .....	5
6 Technical Data .....	6

---

# 1 Description and Intended Purpose

These instructions are for the user.

The term *device* is used throughout these instructions for use to refer to Mini BladeXT.

Make sure that you understand all instructions for use, drawing particular attention to the maintenance and safety information.

## Application

This device is to be used exclusively as part of a lower limb prosthesis.

Intended for a single user.

This device is suitable for walking and is designed specifically for general sport and running.

---

# 2 Safety Information

**This warning symbol highlights important safety information which must be followed carefully.**

Any changes in the performance or function of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The user should be advised that because of the deep 'C' shape side profile (A-P profile) of this device, care should be taken when descending stairs and steps due to the potential hazard of 'over-stepping'.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.

The user should contact their clinician if their condition changes.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

To minimize the risk of slipping and tripping, the sole cover should not be excessively worn, torn or loose.


Be aware of finger trap hazard at all times.

# 3 Construction

## Principal Parts

- Heel & Toe Springs E-carbon
- Heel attachment Screws & Washers Stainless steel, silver
- Sole Cover & Wedge Polyurethane
- Mini Pyramid Titanium
- Pyramid Washer and Nut Stainless steel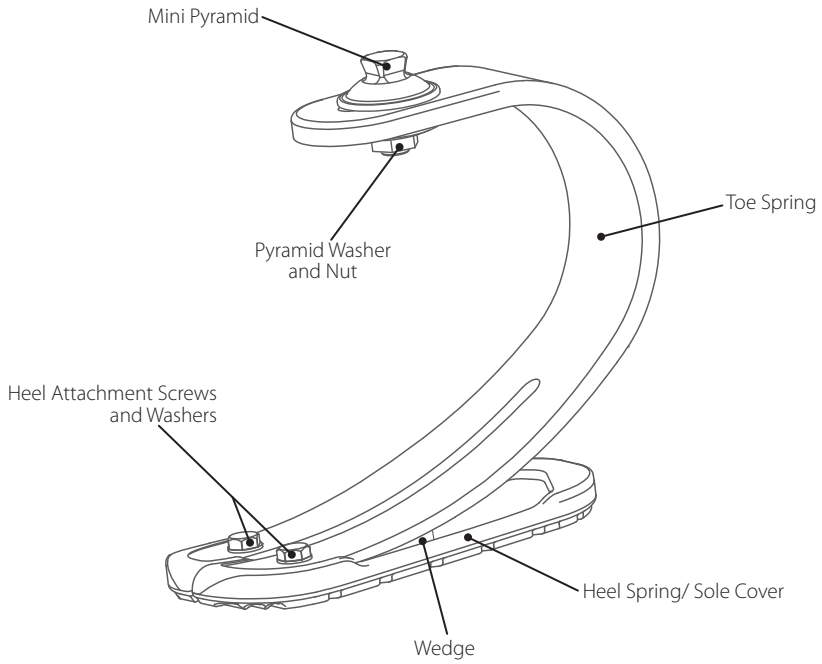

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

We recommend the following annual maintenance:

- Check all screws for tightness, clean and reassemble as necessary.
- Check heel and toe springs for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user should be advised:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.
- Excessive wear of the sole cover could lead to loss of grip. The limb should not be used if the sole cover is excessively worn, torn or loose.
- Excessive wear or damage to the spring elements could lead to structural failure.

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

A regular visual check of the foot is recommended, signs of wear that may affect function should be reported to their service provider.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

---

## 5 Limitations on Use

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15 °C and 50 °C  
(5 °F to 122 °F).

Suitable for submersion


---

## 6 Technical Data

Material:	E-carbon, PU , titanium, St. Stl.
Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight:	450 g (1 lb)
Activity Level:	4
Maximum User Weight:	60 kg (132 lb)
Proximal Alignment Attachment:	Male Mini Pyramid (Blatchford)
Range of Adjustment:	360° rotation ±5° angular 30 mm A-P shift
Build Height: Proximal pyramid to ground level (See diagram below)	168 mm

---

### Fitting Length

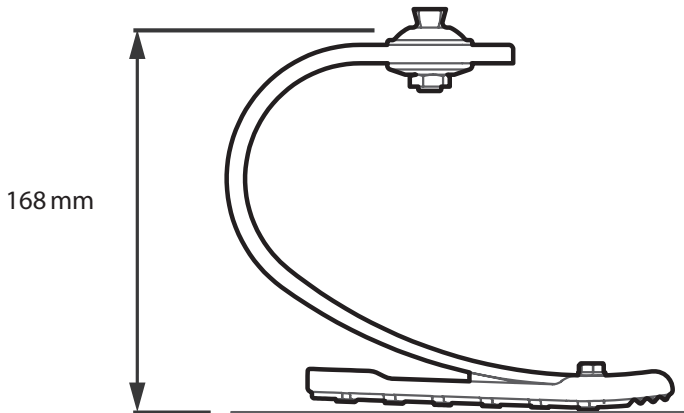## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Warranty

This device is warranted for 12 months (excludes paintwork and sole cover). The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions. See Blatchford website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

See Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

## Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

## Trademark Acknowledgements

BladeXT and Blatchford are registered trademarks of Blatchford Products Limited.

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**Blatchford Europe GmbH**
**Endolite India Ltd.**