# Detachable Cosmetic Cover

## Instructions for Use

561101BLACK

EN	Instructions for Use	2	SK	Návod na použitie	38
FR	Instructions d'utilisation	5	HU	Használati útmutató	41
DE	Gebrauchsanweisung	8	EL	Οδηγίες χρήσης	44
IT	Istruzioni per l'uso	11	LV	Lietošanas pamācība	47
ES	Instrucciones de uso	14	LT	Naudojimo instrukcija	50
NL	Gebbruksaanwijzing	17	ET	Kasutusjuhend	53
PL	Instrukcja użytkowania	20	DA	Bruksanvisning	56
PT	Instruções de utilização	23	NO	Bruksanvisning	59
CS	Návod k použití	26	FI	Käyttöohjeet	62
SL	Navodila za uporabo	29	SV	Bruksanvisning	65
BG	Инструкции за употреба	32	RU	Инструкция протезиста	68
HR	Upute za uporabu	35	ZH	使用说明	71
			AR	إرشادات الاستخدام	74

**Blatchford:**

Contents .....	2
1 Description and Intended Purpose .....	2
2 Safety Information .....	2
3 Function.....	2
4 Maintenance .....	2
5 Limitations on Use .....	2
6 Fitting Advice.....	3
7 Technical Data .....	3

## 1 Description and Intended Purpose

These instructions for use are intended for use by the practitioner and user unless otherwise stated. Please read and ensure you understand all instructions for use, in particular all safety information and maintenance instructions.

The term *device* is used throughout these instructions for use to refer to Detachable Cosmetic Cover.

### Application

This device is to be used exclusively as part of a lower limb prosthesis and is intended for a single user.

The device is a finishing component that is only for use as a cosmetic cover on the following Blatchford devices: Orion3 and Linx.

### Contraindications

The weight of the device should be taken into consideration before fitting (see Section 7, *Technical Data*).

### Clinical Benefits

- Provides an alternative aesthetic appearance without affecting the performance of the knee.

## 2 Safety Information

 **This warning symbol highlights important safety information which must be followed carefully.**

 **Do not expose this device to naked flames.**

 **Report any changes in performance of this device to the practitioner/service provider.**

## 3 Function

The device covers the mechanical components of the Orion3 or Linx limb, to provide a more realistic limb shape under clothing, without inhibiting function of the Orion3 or Linx.

## 4 Maintenance

Visually check the device regularly. Report any changes in performance of this device to the practitioner/service provider (for example, unusual noises or play).

The device has no serviceable parts and must be replaced if damaged.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

Doff the device to clean the interior, then dry thoroughly before donning.







## 5 Limitations on Use

For information on limitations on use, refer to the user guide that corresponds to your knee unit:

Linx	 938354V2A	Orion3	 938375V2A
------	--	--------	---

## 6 Fitting Advice

 Do not open wider than is necessary to don and doff.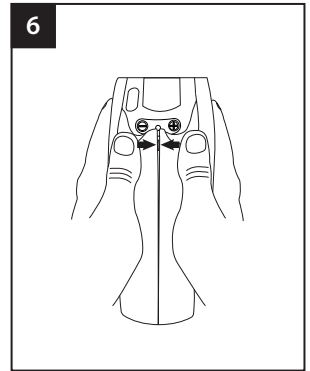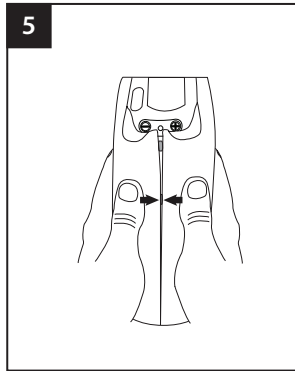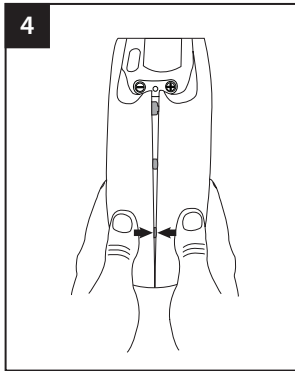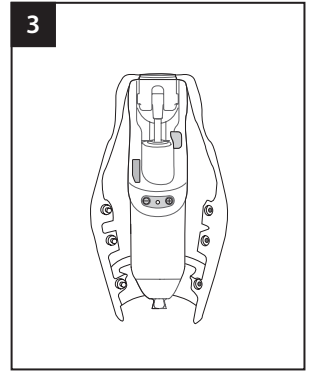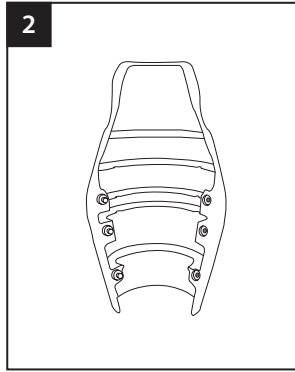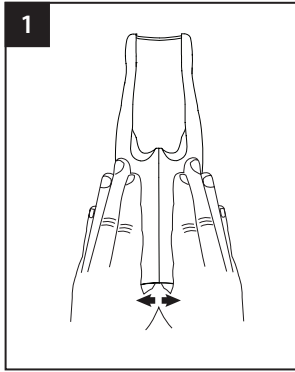

## 7 Technical Data

### Fitting Length

Materials:

Cover

Fasteners

Operating and  
Storage Temperature  
Range:

Component Weight:



Build Height:

Polyurethane  
Aluminum alloy

-15 °C to 50 °C  
(5 °F to 122 °F)

450 g (1 lb)

313 mm

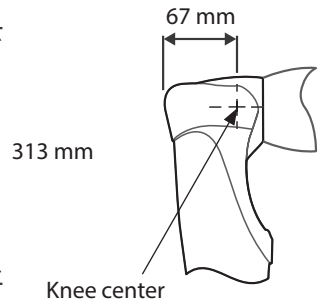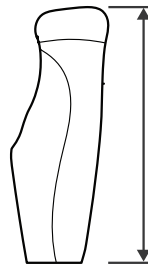## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient –  
multiple use

## Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

## Warranty

This device is warranted for 6 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See the Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

This product is fabricated from a polyurethane that cannot be easily recycled: please dispose of it responsibly as general waste, as per local handling regulations.

## Retaining the Packaging Label

The practitioner is advised to keep the packaging label as a record of the device supplied.

## Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

## Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

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**Blatchford Europe GmbH**
**Endolite India Ltd.**