# Esprit

## User Guide

ESP22L1S–ESP30R8S  
ESP22L1SD–ESP30R8SD

EN	User Guide	2
FR	Guide de l'utilisateur	8
DE	Bedienungsanleitung	14
IT	Guida per il paziente	20
ES	Guía del usuario	26
NL	Gebruikershandleiding	32
PL	Podręcznik użytkownika	38
PT	Guia do utilizador	44
CS	Návod k obsluze	50

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	3
3 Construction .....	4
4 Function.....	5
5 Maintenance .....	5
6 Limitations on Use .....	5
7 Technical Data .....	6

---

# 1 Description and Intended Purpose

**Please keep these instructions.**

The term device is used throughout this document to refer to Esprit.

## Application

This device is to be used exclusively as part of a lower limb prosthesis.

Intended for a single user.

It is a moderate-energy-return foot. Independent heel and toe spring provides some axial deflection. The split toe provides good ground compliance.

---

## 2 Safety Information

**This warning symbol highlights important safety information which must be followed carefully.**

Any changes in the performance or function of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.

The user should contact their clinician if their condition changes.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.




To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

# 3 Construction

## Principal parts:

- Carrier Assembly with Pyramid (aluminum/St. Stl./titanium)
- Heel & Toe Springs (e-carbon)
- Spring attachment screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot shell (PU)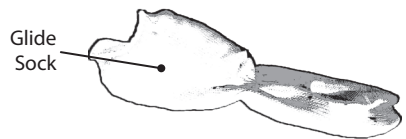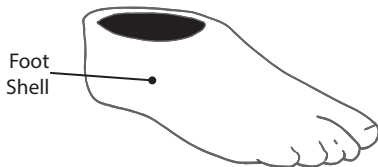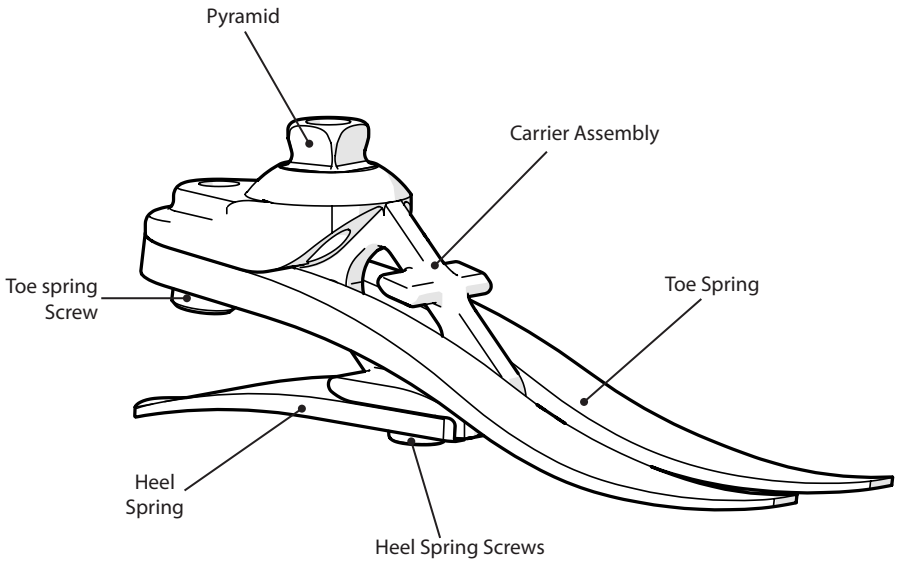

---

## 4 Function

This device comprises an e-carbon toe and independent heel spring. Heel and toe springs are attached to the carrier using titanium and stainless steel screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

---

## 5 Maintenance

Maintenance must be carried out by competent personnel.

We recommend a regular visual check at least annually. Signs of wear that may affect function should be reported to their service provider (e.g. significant wear or excessive discoloration from long term exposure to UV).

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- reduced energy return
- unusual noises
- excessive or reduced movement

The practitioner must also be informed of any changes in body weight and/or activity level.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

---

## 6 Limitations on Use

### Intended life

A local risk assessment should be carried out based upon activity and usage.

### Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).


Suitable for submersion

## 7 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (Size 26N):	530 g (1 lb 3 oz)
Activity Level:	3
Maximum User Weight:	125 kg (275 lb)
Proximal Alignment Attachment:	Male Pyramid (Blatchford)
Range of Adjustment:	±7° angular
Build Height:	(sizes 22–24) 65 mm
(See diagram below)	(sizes 25–26) 70 mm
	(sizes 27–30) 75 mm

### Fitting length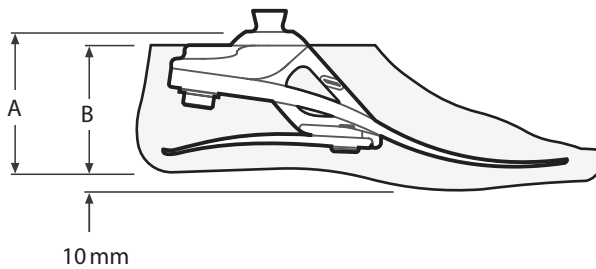

Size	A
22–24	65 mm
25–26	70 mm
27–30	75 mm

Size	B
22–26	65 mm
27–28	70 mm
29–30	75 mm

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Warranty

This device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

## Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

## Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

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**Blatchford Europe GmbH**
**Endolite India Ltd.**