# Detachable Cosmetic Cover

## Instructions for Use

561101BLACK

EN	Instructions for Use	2
FR	Instructions D'Utilisation	3
DE	Gebrauchsanweisung	4
IT	Istruzioni per L'Uso	5
ES	Instrucciones de Uso	6
NO	Bruksanvisning	7
RU	Инструкция протезиста	8
TR	Kullanım Talimatları	10

**Blatchford:**

**EN:** Do not open wider than is necessary to don and doff.

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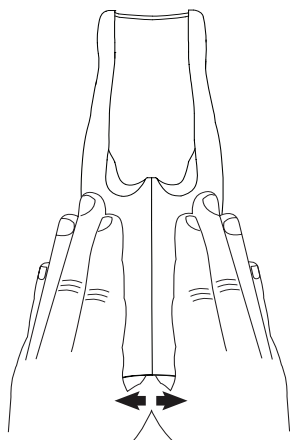
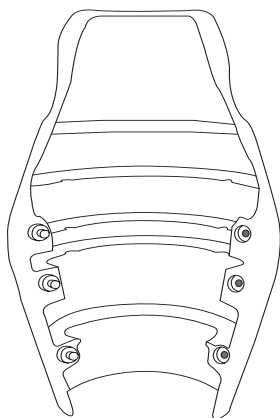
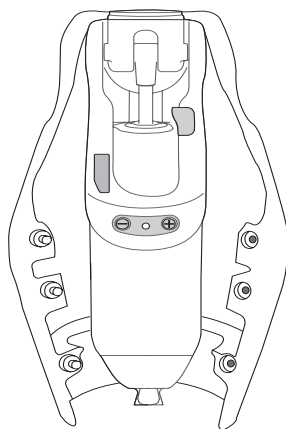
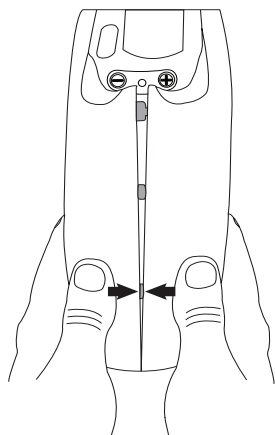
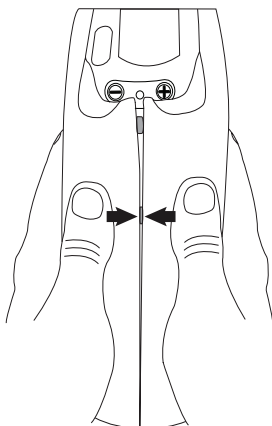
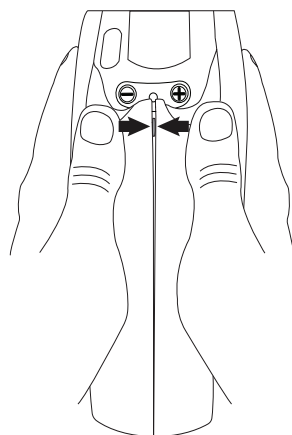
## 1 Description and Purpose

### Application

Make sure that the user understands all the information in these instructions for use, particularly the maintenance information.

This device is a finishing component that is only for use as a cosmetic cover on the following Blatchford devices:

- Orion3
- Linx

### Contra-indications

This device is for a single user.

## 2 Maintenance

### Environment

Advise the user to report any changes in the performance of this device to their clinician. Changes in performance include unusual noise or play.

Do not expose this device to naked flames.

For more information about environmental limitations on use, refer to *Key Points* in the user guide that corresponds to your knee unit:

Linx	 9383545
Orion3	 938375

### Cleaning

Use a damp cloth and a mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

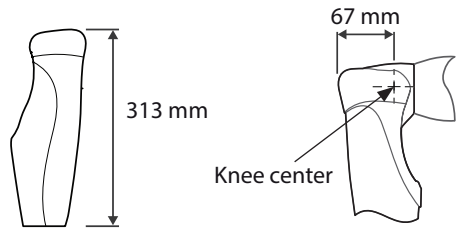
Doff the device to clean the interior, then dry thoroughly before donning.

## 3 Technical Data

Component Weight: 425 g (15 oz)

Material: Polyurethane

### Fitting Length

### Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

### CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

### Warranty

This device is warranted for 6 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

### Environmental Aspects

This product is fabricated from a polyurethane that cannot be easily recycled: please dispose of it responsibly as general waste, as per local waste handling regulations.

### Manufacturer's Registered Address

Blatchford Products Limited, Lister Road,  
Basingstoke RG22 4AH, UK.

Blatchford is a trademark of Blatchford Products Ltd.

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Blatchford SAS
Endolite India Ltd